**Speed Trio FeLV/FIV/Corona™**

# Speed Trio FeLV/FIV/Corona™

## Veterinary diagnostic kit

### For *in vitro* use only

#### ■ CLINICAL APPLICATION

Feline Leukaemia virus (FeLV), Feline Immunodeficiency virus (FIV), and Feline Coronavirus (FCoV) infections are considered as being major causes of fatal infectious diseases in the cat worldwide. Transmission of these viruses is primarily through close contact between cats (saliva, nasal secretions, faeces, etc.).

FeLV, FIV, and FCoV infections can present many clinical similarities; the differential diagnosis can therefore be a challenge:

- Symptoms are changeable and often unspecific.
- Sick cats present with very similar clinical symptoms: depression, anorexia, pyrexia, immunodeficiency syndrome, anaemia, etc.

#### ■ PRINCIPLE

Speed Trio FeLV/FIV/Corona is a rapid qualitative immunoassay, using immunochromatography, for the detection of FeLV antigens, and anti-FIV and anti-FCoV antibodies in cats.

For each test, one drop of the sample (serum, plasma, or anticoagulated whole blood) is added to the sample well. In the sample well, the stained particles of the conjugate bind to any FeLV P27 antigens (FeLV part), anti-FIV antibodies (FIV part) and anti-FCoV antibodies (FCoV part) present in the sample. The resulting conjugate/FeLV antigen, conjugate/anti-FIV antibody, and conjugate/anti-FCoV antibody complexes migrate along the membrane via capillarity. These complexes are captured by specific complex binders that are coated to the membrane, forming a pink test band by accumulation of coloured particles. The mixture continues to migrate along the strip to the end of the membrane where the remaining coloured particles form a pink control band, which confirms the validity of the test.

## ■ PROCEDURE

### ► FOR EACH TEST YOU WILL NEED:

1 test device, 1 single-use pipette and the reagent dropper bottle.

The reagents should be used at room temperature.

**Do not mix reagents from different batches.**

### 1/ ADD THE SAMPLE:

Serum, plasma, anti-coagulated whole blood (EDTA or heparin): with a single-use pipette held vertically, **add 1 drop** of sample to each sample well.

### 2/ ADD THE REAGENT:

- Add the reagent immediately after the sample. Hold the dropper bottle in a vertical position and **add 5 drops** of reagent to each sample well.

- If no migration is observed within 2 minutes, **add an extra 2 drops** of reagent to the sample well.

### 3/ READING AND INTERPRETATION OF RESULTS:

**Wait 15 minutes** for migration and read:

- **NEGATIVE TEST:** any test showing only **1 pink band** (control band) is a negative result.

- **POSITIVE TEST:** any test showing **2 distinct pink bands** (Test band + Control band) indicates a positive result.

The test is positive if a pink test band appears within 10 minutes of migration. Any colour change in the test band, even faint, should be considered as a positive result.

- **If no control band appears, the test is invalid.**

### • **INTERPRETATION FOR CORONAVIRUS ANTIBODY TEST:**

A positive result for FCoV test indicates previous contact with feline coronavirus. FCoV seropositive cats may excrete the virus in their faeces.

In sick cats, when used in combination other diagnostic tests, coronavirus antibody test provides an indication of the likelihood of the cat suffering from FIP. However, towards the end of the disease process, the formation of immune complexes can reduce circulating antibody levels resulting in false negative results.

## ■ **RECOMMENDATIONS**

### • **SHELF LIFE:**

- 24 months at room temperature (from manufacturing date). The expiry date is printed on each box and on each test pouch.
- Store at room temperature, between +2°C to +30°C. Keep the test away from sources of excessive heat or cold.

### • **SAMPLES:**

- Serum, plasma, or anticoagulated whole blood must be used at room temperature.
- Effusion fluid is not validated for FeLV and FIV testing with Speed Trio FeLV/FIV/Corona and should not be used for FeLV antigen detection or anti-FIV antibody detection
- If testing within 24 hours after sampling: store serum, plasma, or anticoagulated whole blood at room temperature.
- If testing between 24 to 72 hours of sampling: store serum or plasma at between +2°C and +8°C (refrigerator). Storing whole blood between +2 and +8°C for several hours may induce haemolysis (not visible to the naked eye), which may interfere with the interpretation of the results.
- If testing more than 72 hours after sampling: store serum or plasma at -20°C. Never freeze a whole blood sample.

### • **PRECAUTIONS:**

- To ensure correct sample migration, place the test device on a flat

and horizontal surface.

- To standardize the amount of sample recommended for each test, only use the single-use pipettes supplied in the kit.
- To ensure that the sample and reagent are correctly applied, hold the pipette or dropper bottle vertically.

• **OTHER RECOMMENDATIONS:**

- When performing the test, the reading window may show a light pink background coloration; this does not affect the quality of the results.
- **Do not mix reagents from different batches.**
- Allow the reagents to come to room temperature before use.
- Use a new single-use pipette for each test.

These recommendations are only guidelines; no test is 100% accurate at all times and under all conditions. The purpose of this test is to detect FeLV antigens, anti-FIV antibodies, and anti-Coronavirus antibodies in cats. All test results should be interpreted in the light of the patient's history, clinical examination, and the results of any further diagnostic tests. The definitive diagnosis remains the veterinarian's prerogative and responsibility.

Bio Veto Test and its distributors cannot be held responsible for the consequences of misuse or misinterpretation of the results given by the test.

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