Speed Giardia™

English	1
Français	5
Español	9
Português	13
Italiano	17
Ελληνικά	21
Deutsch	25
Netherlands	29
Procedure	33

FOR *IN VITRO* USE ONLY ENGLISH

■ CLINICAL APPLICATION

Giardiasis is a protozoosis that infests several different species including dogs, cats, and cattle. *Giardia duodenalis* (also known as *Giardia lamblia*) is a flagellated parasite with two different morphologic stages: a reproductively active intestinal form, or trophozoite, and an inactive cystic form in which it is excreted by the host.

Contamination occurs through the ingestion of cysts in contaminated food or water or via the faecal-oral route, and via contact with others animals. Clinical symptoms depend on the degree of infestation, species, and age of the animal. Symptoms include acute or chronic diarrhoea, intestinal malabsorption with growth retardation, and eating disorders with loss of appetite.

Testing for Giardiasis should only be conducted in cases with a strong clinical suspicion, since asymptomatic carriers are common. Cysts are excreted within an average of 7 days of infestation. Oocyst excretion is intermittent, which is why it is important to test overall several days, in order to make the diagnosis at the peak of excretion. The numbers of cysts that are excreted during an infestation varies between individuals, as a function of the intermittency of excretion, and the counting technique used.

Clinical and autopsy findings are non-specific, the only reliable diagnostic technique is a faecal count. Parasitological screening is recommended in young animals - dogs and cats aged between 6 weeks and 3 months, calves aged between 1 week and 6 months. It can also be useful in older animals to diagnose the cause of chronic diarrhoea or to detect carriers, which act as a source of contamination.

■ PRINCIPLE

Speed Giardia is a rapid qualitative immunoassay strip test using

immunochematography for the detection of soluble antigens of *Giardia duodenalis* in faecal samples from dogs, cats, and calves.

For each test, in the sample well, the stained particles of the conjugate bind to any *Giardia* antigens present in the sample. The resulting conjugate/antigen complexes migrate along the membrane via capillarity. They are captured by specific *anti-Giardia duodenalis* antibodies that are bound to the membrane, forming a red test band by accumulation of coloured particles. The mixture continues to migrate to the end of the strip where the remaining coloured particles form a blue control band, which confirms the validity of the test.

■ PROCEDURE

► FOR EACH TEST YOU WILL NEED:

1 bag with strip test, 1 sample tube with buffer diluent and “sampling spoon” and 1 test holder.

The reagents should be used at room temperature.

Do not mix reagents and strips from different batches.

1/ ADD THE SAMPLE:

- Label the sample tube with the animal's name. Take **one level sampling spoonful** of faeces and add it to the buffer diluent. Close the vial and shake to homogenize.

- Allow the solution to sediment for **3 minutes**.

2/ PERFORM THE TEST:

- Take the strip out of its bag and gently plunge it into the solution in the direction indicated by the arrow, without immersing the central reactive band.

- Allow the strip to stand for **1 minute in the solution**.

- **Remove the strip** and place it on a **flat, horizontal surface**.

- Leave to migrate.

3/ READING AND INTERPRETATION:

READING:

Wait **5 minutes for the liquid to migrate** and read:

- **NEGATIVE TEST: a single blue band** appears (control band).

- **POSITIVE TEST: a red band and a blue band** appear (test band + control band).

Any red coloration of the test band (even a faint red coloration) should be considered as a positive result.

- **If no blue control band appears, the test is invalid.**

INTERPRETATION:

Speed Giardia is a qualitative test to detect the presence of Giardia antigens in the faeces.

If the animal does not present any clinical symptoms at the time of sampling, a positive animal should be considered as a healthy carrier, and thus a source of contamination in group housing situations. Due to intermittent excretion, if a negative result is obtained despite clinical suspicion, it is advisable to perform repeated Speed Giardia tests, 3 times at 48 hour intervals, or conduct tests on several animals from the same breeding unit.

RECOMMENDATIONS

• SHELF LIFE:

- 16 months at room temperature (from manufacturing date). The expiry date is printed on each box and on each test pouch.
- Store at room temperature, between +2°C to +30°C. Keep the test away from excess heat; do not freeze.

- After opening the strip bag, do not expose the strip to humidity before use and do not touch the central reactive zone.

• **STORAGE OF SAMPLES:**

- Speed Giardia must be **run as soon as possible after sampling.**
- Take **a level spoonful of faeces.**

• **PRECAUTIONS:**

- Do not touch the central reactive zone of the strip.
- To ensure correct migration of the sample, place the test strip on a flat, horizontal surface.

• **OTHER RECOMMENDATIONS:**

- **Do not mix reagents and test strips from different batches.**
- Allow the reagents and strips to come to room temperature before use.
- Use a new sampling spoon for each test.

The above recommendations are only guidelines; no test is 100% accurate at all times and under all conditions. The purpose of this test is to detect the soluble antigens of *Giardia duodenalis* to diagnose Giardiasis. All test results should be interpreted in the light of the patient's history, clinical examination, and the results of any further diagnostic tests. The definitive diagnosis remains the veterinarian's prerogative and responsibility.

Bio Veto Test and its distributors cannot be held responsible for the consequences of misuse or misinterpretation of the results given by the test.

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