Speed[™] **Diro**

Speed™ Diro

Veterinary diagnostic kit
For *in vitro* use only

■ CLINICAL APPLICATION

Cardiopulmonary *Dirofilariasis*, or heart worm, is a helminthiasis caused by the presence and development of a parasite called *Dirofilaria immitis* in the pulmonary arteries and right ventricle. This parasite infects dogs and sometimes cats and is transmitted by mosquito bites.

Clinical signs include coughing, listlessness, tachypnoea, heart murmur, and dyspnea. Serological testing for circulating antigens of adult filariae is a highly sensitive method of confirming the diagnosis. The detection of circulating antigens also enables the detection of occult filariasis (without microfilariae: 30 % of cases of filariasis).

Speed Diro has the following indications:

- confirm a suspicion of Heartworm infestation by screening for *Dirofilaria immitis* antigens indicating the presence of adult worms in the circulatory system.
- implementing prophylactic treatment.
- initiating and monitoring prophylactic treatment.

■ PRINCIPLE

Speed Diro is a rapid qualitative immunoassay, using immunochromatography, for the detection of the circulating antigen of *Dirofilaria immitis*.

EN-1

For each test, one drop of sample (serum, plasma or anti-coagulated whole blood) is added to the sample well. In the sample well, the stained particles of the conjugate bind to any circulating antigens of *Dirofilaria immitis* present in the sample. The resulting conjugate/antigen complexes migrate along the membrane via capillarity. They are captured by specific antibodies that are bound to the membrane, forming a pink test band by accumulation of coloured particles. The mixture continues to migrate along the strip to the end of the membrane where the remaining coloured particles form a pink control band, which confirms the validity of the test.

■ PROCEDURE

► FOR EACH TEST YOU WILL NEED:

1 test device, 1 single-use pipette, and the reagent dropper bottle.

The reagents should be used at room temperature

Do not mix reagents from different batches

1/ ADD THE SAMPLE:

Serum, plasma, or anti-coagulated whole blood (EDTA or heparin) :

Using the provided single-use pipette held vertically, **add 1 drop** of sample **at the centre of the sample well and wait 10 to 15 seconds.**

2/ ADD THE REAGENT:

- **Add 5 drops of reagent** to the sample well, holding the dropper bottle in a vertical position.

- If no migration is observed within 2 minutes, **add an extra 2 drops** of reagent to the sample well.

3/ READING AND INTERPRETATION OF RESULTS:

Wait 15 minutes for migration before reading:

- **NEGATIVE TEST:** any test showing only **1 pink band** (control band) is a negative result.

- **POSITIVE TEST:** any test showing **2 distinct pink bands** (test band + control band) indicates a positive result. The test is positive if a pink test band appears within 10 minutes of migration. Any colour change in the test band, even faint, should be considered as a positive result.

- **If no control band appears, the test is invalid.**

■ RECOMMENDATIONS

• SHELF LIFE:

- 24 months at room temperature (from manufacturing date). The expiry date is printed on each box and on each test pouch.
- Store at room temperature, between +2°C to +30°C. Keep the test away from sources of excessive heat or cold.

• SAMPLES:

- Serum, plasma, or anti-coagulated whole blood must be used at room temperature.
- For testing within 24 hours of sampling: store serum, plasma, or anti-coagulated whole blood at room temperature.

EN-3

- For testing between 24 to 72 hours of sampling: store serum or plasma between +2°C and +8°C (fridge). Whole blood storage between +2°C and +8°C for several hours may induce haemolysis (not visible to the naked eye), which may interfere with the interpretation of the results.
- Testing more than 72 hours after sampling: store serum or plasma at -20°C.

• **PRECAUTIONS:**

- To ensure correct migration of the sample, place the test device on a horizontal surface.
- To standardize the amount of sample recommended for each test, only use the single-use pipettes supplied in the kit.
- To ensure that the sample and reagent are correctly applied, always hold the pipette or dropper bottle vertically.

• **OTHER RECOMMENDATIONS:**

- The reading window may show a light pink background coloration; this does not affect the quality of the results.
- **Do not mix reagents from different batches.**
- Allow the reagents to come to room temperature before use.
- Use a new single-use pipette for each test.

The above recommendations are only guidelines; no test is 100% accurate at all times and under all conditions. The purpose of this test is to detect *Dirofilaria immitis* antigens to diagnose Dirofilariasis. All test results should be interpreted in the light of the patient's history, clinical examination, and the results of any further diagnostic tests. The definitive diagnosis remains the veterinarian's prerogative and responsibility.

Bio Veto Test and its distributors cannot be held responsible for the consequences of misuse or misinterpretation of the results given by the test.

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SPEED and *Speed* are trademarks of Virbac or its affiliates